First Sign®
One-Step Pregnancy Test (Strip)
For Self-testing Use Only

INTENDED USE

First Sign® One-Step Pregnancy Test is a rapid and convenient chromatographic immunoassay designed for the qualitative determination of human chorionic gonadotropin (hCG) in urine for early detection of pregnancy. The test strip is intended for self-testing.

INTRODUCTION

Human chorionic gonadotropin (hCG) is a glycoprotein hormone secreted by the developing placenta shortly after fertilization. During normal pregnancy, hCG can be detected in urine as early as 7 days following conception, doubling every 1.3 to 2 days. At the time of the last missed menstrual period, urine hCG levels are about 100mIU/mL with peak levels of 100,000 to 200,000mIU/mL seen at the end of the first trimester. The presence of hCG soon after conception and its subsequent increase in concentration during early gestational growth make it an ideal marker for the early detection of pregnancy.

MATERIALS

Material Provided:

- Urine collection cup
- First Sign® One-Step Pregnancy Test strip individually sealed in a foil pouch.
- Each pouch contains:
 1. One First Sign® One-Step Pregnancy Test strip
 2. Desiccant

Material Required, But Not Provided:

1. Timer
 2. Disposable gloves
- No other equipment or reagents are needed.

STORAGE AND STABILITY

Store test strip at 4°C to 30°C (40°F to 86°F). Do not freeze. The test is stable until the date imprinted on the pouch label.


WARNINGS AND PRECAUTIONS

1. For *in vitro* diagnostic use only.
2. For self-testing use only.
3. Read directions for use carefully before performing this test.
4. Do not use the test beyond the labeled expiration date.
5. Do not use if pouch is damaged or opened.
6. Do not reuse the test strip. Discard it in the trash can after single use.
7. Do not touch the membrane located within the windows.
8. Use immediately after opening the pouch. Prolonged exposure to ambient humidity will cause product deterioration.
9. Treat urine specimen and used strips as potentially infectious. Avoid contact with skin.
10. Avoid internal use and children touching.
11. It is recommended that you reduce water intake prior to testing. Water intake can dilute your urine sample and may affect the detection of hCG.
12. Drinking alcohol during pregnancy may be harmful to the fetus.

PROCEDURE

1. **Determination of Test Date**
The test can be used on the first day of a missed period.
2. **Specimen Collection and Handling**
The First Sign® One-Step Pregnancy Test is formulated for use with fresh urine specimens. The test should be used right after the specimen collection. Urine specimen does not require any special pretreatment.
3. **Test Procedure**
 - 1.) Urinate into enclosed cup. If you prefer, you may use any other disposable cup.
 - 2.) Remove test strip from its foil pouch by tearing along the slice.


- 3.) Immerse the strip into the urine with the arrow end pointing toward the urine. Do not cover the urine over the MAX (maximum) line. You may take the strip out after minimum of 15 seconds in the urine and lay the strip flatly on a non-absorbent clean surface.
- 4.) Read result at five minutes. **DO NOT INTERPRET RESULT AFTER TEN MINUTES.**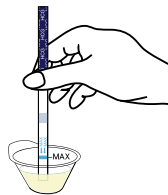

4. **Interpretation of Results**


Negative

If only one line appears in the Control region, you can assume that you are not pregnant.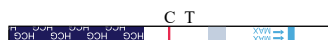


Positive

If two lines appear, one in the Control region and another one in Test region, you can assume that you are pregnant.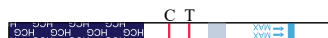

Invalid

Control line fails to appear.
NOTE: Insufficient specimen volume or incorrect procedural techniques are the most likely reasons for control line failure. Review the procedure and repeat the test with a new test strip. If the problem persists, please contact your local distributor.

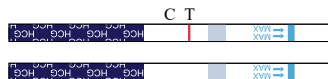QUALITY CONTROL

The procedural control is included in the test. A colored line appearing in the Control region (C) is considered an internal procedural control, indicating sufficient specimen volume and correct procedural technique.

LIMITATIONS

1. Alcohol may interfere with the test result. It is not recommended using the test after drinking.
2. Occasionally specimens containing less than 25mIU/mL hCG for urine also yield positive results.
3. A very early pregnancy containing an extremely low concentration of hCG can give a negative result. In this case, another specimen should be obtained at least 48 hours later and tested.
4. hCG levels may remain detectable for several weeks after normal delivery, delivery by caesarean section, spontaneous abortion or therapeutic abortion.
5. In cases where very high levels of hCG are present (>500,000mIU/mL) a false negative result can occur due to "Prozone" effect. If pregnancy is still suspected, simply dilute specimen 1:1 with deionized water and retest.
6. If a urine specimen is too diluted (i.e.: low specific gravity) it may not contain a representative level of hCG. If pregnancy is still suspected, a first morning urine sample should be obtained from the user in 24-48 hours and retested.
7. As is true for any diagnostic procedure, the user should evaluate data obtained by the use of this test in light of other clinical information and consult with health care professional for the final diagnosis of pregnancy.

QUESTIONS AND ANSWERS

1. Q: How does the test work?
A: The First Sign® One-Step Pregnancy Test detects human chorionic gonadotropin (hCG) which is a glycoprotein hormone secreted by the developing placenta shortly after fertilization. The amount of hCG levels increases as pregnancy progresses.
2. Q: How soon after I suspect that I am pregnant can I take the test?
A: You can test your urine as early as the first day you miss your period. You can perform the test anytime of the day; however, if you are pregnant, first morning urine specimen contains the highest concentration of hCG.
3. Q: Do I have to test with first morning urine?
A: Although you can test at any time of the day, your first morning urine specimen is usually the most concentrated of the day and would have the most hCG in it.
4. Q: How accurate is the test?
A: In both laboratory and consumer clinical studies, the First Sign® One-Step Pregnancy Test has been proven to be greater than 99% accurate.
5. Q: Which factors may affect the result?
A: Drugs, which contains hCG such as Pregnyl, Profasi, Pergonal, or APL, can give a false positive result. Alcohol, oral contraceptives, painkiller, antibiotics or hormone therapies that do not contain hCG should not affect the test result.
6. Q: What should I do if the result shows that I am pregnant?
A: It means that your urine contains hCG and you are probably pregnant. See your doctor to confirm that you are pregnant and to discuss the steps you should take.
7. Q: Does the result mean anything other than normal pregnancy if it shows that I am pregnant?
A: A number of medical conditions other than pregnancy, includes ovarian cyst or ectopic pregnancy (pregnancy outside the uterus) can cause elevated levels of hCG.
8. Q: How do I know that the test was run properly?
A: The appearance of a line in the control window (C) tells you that you had followed the test procedure properly and the proper amount of urine was absorbed.
9. Q: What should I do if the result shows that I am not pregnant?
A: It means that no hCG has been detected in your urine and probably you are not pregnant. If you do not start your period with a week of its due date, repeat the test with a new test strip. If you receive the same result after repeating the test and you still do not get your period, you should see your doctor.

Questions or comments? Call toll free 1-888-HEMOSURE (436-6787)
Monday – Friday from 8:00 a.m. – 5:00 p.m., Pacific Time

Manufactured for:
Hemosure, Inc.
5358 Irwindale Ave.
Irwindale, CA 91706
www.hemosure.com

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