

# Ovultell®

## Ovulation Test

One-Step Ovulation (+) Pregnancy Tests

The **Ovultell® One-Step Ovulation Test** is a rapid and convenient chromatographic immunoassay designed for the qualitative determination of human luteinizing hormone (hLH) in urine to predict the time of ovulation. The test strip is intended for self-testing.


The **First Sign® One-Step Pregnancy Test** is a rapid chromatographic immunoassay designed for the qualitative determination of human chorionic gonadotropin (hCG) in urine for early detection of pregnancy. The test strip is intended for self-testing.

### Easy to Use

- Result at 5 minutes
- Clear & accurate result
- Built-in quality control



Ovultell®  
LH Strip

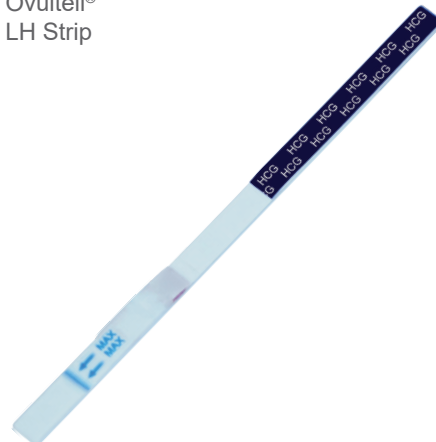

First Sign®  
hCG Strip



Call us for more information!

**888.HEMOSURE (436.6787)**

# Ovulation

## How to Read Result

### NEGATIVE

If only Control line appears, or the Test line is much lighter than the Control line, the test result is negative which suggests no LH surge has been detected.


### POSITIVE

If both Test line and Control line are visible and the Test line is darker than or equal to the Control line, the test result is positive which means LH surge is detected and the ovulation shall occur in the following 24-36 hours.


### INVALID

If there is no distinct pink-purple colored band visible in Control region, the test is invalid.

## Test Procedure

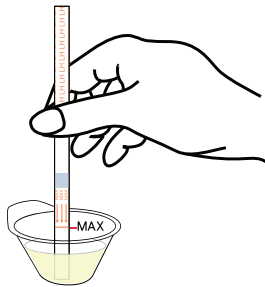

READ THE RESULT AT 5 MINUTES.  
DO NOT INTERPRET RESULT AFTER 10 MINUTES.

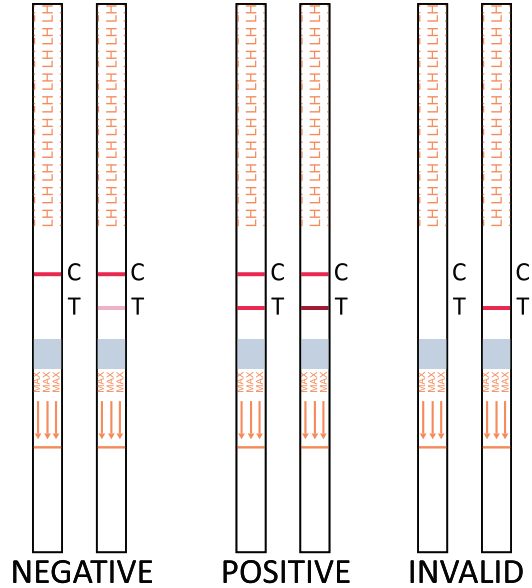

# Pregnancy

## How to Read Result

### NEGATIVE

You are probably not pregnant if only one pink line appears in the control region (C).


### POSITIVE

You are probably pregnant if two pink lines appear, one in the test region (T) and one in the control region (C).


### INVALID

The result is invalid if no line appears in the control window (C), even if a line appears in the test region (T).

## Test Procedure

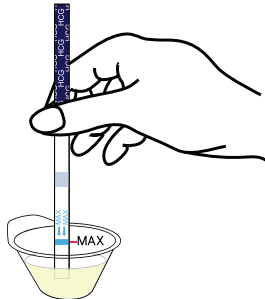

READ THE RESULT AT 5 MINUTES.  
DO NOT INTERPRET RESULT AFTER 10 MINUTES.

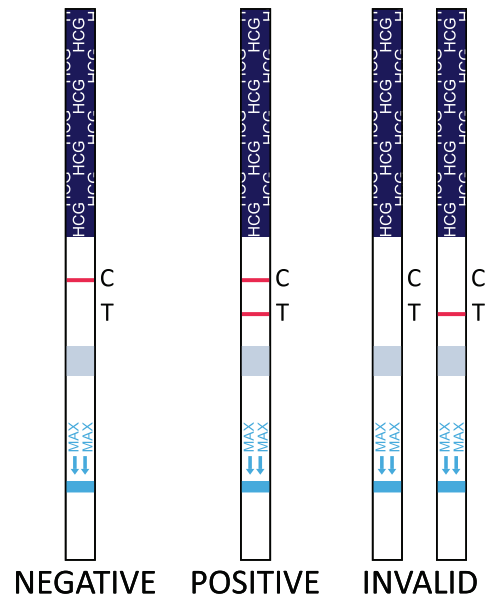

**888.HEMOSURE (436.6787)**

5358 Irwindale Ave., Irwindale, CA 91706 | [www.hemosure.com](http://www.hemosure.com)

© 2017 Hemosure logo is a registered trademark of Hemosure, Inc. All rights reserved.

DGM36109-1.0