Ovutell®
One-Step Ovulation Test (Strip)
For Self-testing Use Only

INTENDED USE

The Ovutell® One-Step Ovulation Test is a rapid and convenient chromatographic immunoassay designed for the qualitative determination of human luteinizing hormone (hLH) in urine to predict the time of ovulation. The test strip is intended for self-testing.

INTRODUCTION

Ovulation is the release of an egg from the ovaries. A low background level of Luteinizing Hormone (LH) is always present in your urine. At about the middle of your monthly cycle, the LH increases suddenly to a high surge level that causes the release of an egg from your ovary (ovulation). To become pregnant, the egg must be fertilized with a sperm cell in time. Therefore, your most fertile time is when you are ovulating. This test can help you predict your ovulation time quickly and easily.

MATERIALS

- Material Provided:**
Urine collection cup
Ovutell® One-Step Ovulation Test strips sealed in a canister.
Each canister contains:
1. Twenty Ovutell® One-Step Ovulation Test strips
2. Desiccant

Material Required, But Not Provided:

1. Timer
2. Disposable gloves
No other equipment or reagents are needed.

STORAGE AND STABILITY

Store test strip at 4°C to 30°C (40°F to 86°F). Do not freeze. The test is stable until the date imprinted on the canister label. Once the canister is opened the test strips are stable for 30 days.

WARNINGS AND PRECAUTIONS

- For *in vitro* diagnostic use only.
- For self-testing use only.
- Read directions for use carefully before performing this test.
- Do not use the test beyond the labeled expiration date.
- Do not open the canister until the first day you test. Once the canister is opened the strips must be used within 30 days.
- Do not reuse the test strip. Discard it in the trash can after single use.
- Do not use if canister is damaged or opened.
- Do not touch the membrane located within the windows.
- Use immediately after opening the canister. Prolonged exposure to ambient humidity will cause product deterioration.
- Treat urine specimen and used strips as potentially infectious. Avoid contact with skin.
- Avoid internal use and children touching.
- Examine if the urine collection cup is dry and clean before usage.

ASSAY PROCEDURE

- Determination of Test Date**
First you must determine your usual length of menstrual cycle. The length of menstrual cycle is the number of days from the first day of menstrual bleeding to the day before bleeding begins on the next period. Determine the usual length of the menstrual cycle over the last few months. Then refer to the Cycle Chart to determine on which day of the menstrual cycle to begin testing. Suggested test for 10 days.
Cycle Chart

Menstrual cycle length	22	23	24	25	26	27	28	29	30	31	32	33	34	35	36	37	38	39
Days to count ahead	5	6	7	8	9	10	11	12	13	14	15	16	17	18	19	20	21	22

Note: If you are uncertain about the average cycle length, you may want to use the shortest length (22 days) indicating on the chart.


2. Specimen Collection and Handling

The Ovutell® One-Step Ovulation Test is formulated for use with fresh urine specimens. The test should be used right after the specimen collection. The urine does not require any special pretreatment. Try to collect urine at the same time each day for the entire cycle. For best results, collect between 10:00am and 8:00pm. You should limit your fluid intake for about two hours before you perform the test.

3. Test Procedure

Note: Before you perform the test for the first time, write the date on the canister label where indicated. Once the canister is opened, the remaining test strips must be used with 30 days when stored in a securely closed canister in a cool, dry place.


- Urine into enclosed cup. If you prefer, you may use any other disposable cup.
- Remove the test strip from the canister and immediately put back the cap on the canister, close tightly.
- Immerse the strip into the urine with the arrow end pointing toward the urine. Do not cover the urine over the MAX (maximum) line. You may take the strip out after minimum of 15 seconds in the urine and lay the strip flatly on a non-absorbent clean surface.
- Read the result at five minutes. DO NOT INTERPRET RESULT AFTER TEN MINUTES.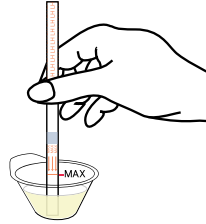

4. Interpretation of Results


Negative

If only Control line appears; or the Test line is much lighter than the Control line, the test result is negative which suggests no LH surge has been detected.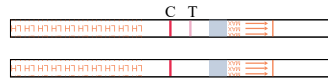

Positive

If both Test line and Control line are visible and the Test line is darker than or equal to the Control line, the test result is positive which means LH surge is detected and the ovulation shall occur in the following 24-36 hours.

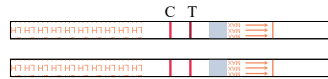Invalid

If there is no distinct pink-purple colored band visible in Control region, the test is invalid. It is recommended that in this case the test needs to be repeated with a new test strip.

QUALITY CONTROL

The procedural control is included in the test. A colored line appearing in the Control region (C) is considered an internal procedural control, indicating sufficient specimen volume and correct procedural technique.

LIMITATIONS

- Alcohol may interfere with the test result. It is not recommended using the test after drinking.
- Women suffering from polycystic ovary syndrome (PCOS) may have elevated LH concentration which may interfere with the test result.
- Pregnant woman should not use this test. Since urine of pregnant women contains high concentration of hCG and reagents used in the test are cross-reactive with hCG.
- As is true for any diagnostic procedure, the user should evaluate data obtained by the use of this test in light of other clinical information and consult health care professionals for advice on when to try to become pregnant.

QUESTIONS AND ANSWERS

- Q:** How accurate is the Ovutell® One-Step Ovulation Test?
A: A clinical evaluation was conducted comparing the results obtained using the Ovutell® One-Step Ovulation Test to another commercially available urine membrane LH test. The results demonstrated an accuracy of Ovutell® One-Step Ovulation Test is more than 99% when compared to the other test.
- Q:** How sensitive is Ovutell® One-Step Ovulation Test?
A: The Ovutell® One-Step Ovulation Test detects luteinizing hormone (LH) in urine at concentrations of 25mU/ml or higher.
- Q:** Do alcohol or common medications affect the test?
A: No, but you should consult your doctor if you are taking any hormonal medication. Also, recent oral contraceptive use, breastfeeding, or pregnancy could affect the test results.
- Q:** Why should I not use first morning urine and what time of the day should I perform the test?
A: We do not recommend first morning urine because it is concentrated and may give a false positive result. Any other time of day is suitable. For best results try and collect your urine at approximately the same time each day.
- Q:** Will the amount of liquid I drink affect the test result?
A: Heavy intake of fluids prior to testing will dilute the hormone in your urine. We suggest that you limit your fluid intake for about two hours before you perform the test.
- Q:** How long will the lines remain visible?
A: The test should be read at five minutes for best results. A positive (surge) result will never disappear. The colored line (s) may become darker and a tinted background may appear after several hours. Some negative results may later display a faint second color line because of evaporation from the test line. Therefore you should not read the results after 10 minutes, discard the test once you have read the result.
- Q:** Once I see a positive result, when is the best time to have intercourse to try and conceive?
A: Ovulation is likely to occur within 24-36 hours. This is your most fertile time. Sexual intercourse within this time frame is advised.
- Q:** I am now using the basal body temperature method (BBT)? Does this test replace the BBT?
A: The test does not replace the BBT method. The shift in basal body temperature primarily indicates that ovulation has already occurred. The test indicates that ovulation is about to occur.
- Q:** I received a positive result and had intercourse during these fertile days but have not become pregnant. What shall I do?
A: There are many factors that can affect your ability to become pregnant. It can take normal, healthy couples many months to achieve a pregnancy and often you may need to use the test kit for 3-4 months before achieving pregnancy. If pregnancy is not achieved after 3-4 months, you and your partner should consult a doctor.
- Q:** Can I use the Ovutell® One-Step Ovulation Test to avoid pregnancy?
A: No, the test shouldn't be used as a form of birth control.
- Q:** I have had a positive result and had intercourse during these fertile days. I think I may be pregnant. How soon can I find out?
A: The First Sign® One-Step Pregnancy Test products can provide you with results as early as the first day you missed your period.
- Q:** Why it is necessary to test twice a day?
A: Since current study shows that 38 % of women have the length of the LH surge only lasting less than 10 hours, we recommend you use Ovutell® One-Step Ovulation Test twice a day in the morning and the evening, to more effectively and accurately catch your ovulation timing. LH surge usually comes before ovulation, which means release of egg. Egg can only live for 24 hours or less after it is released, while sperm survives only a few days in your body. Consequently, the Ovutell® One-Step Ovulation Test can help you predict the two most fertile days following the detected LH surge. Having intercourse within these days will maximize your chances of becoming pregnant.

Questions or comments? Call toll free 1-888-HEMOSURE (436-6787)
Monday – Friday from 8:00 a.m. – 5:00 p.m., Pacific Time

Manufactured for:
Hemosure, Inc.
5358 Irwindale Ave.
Irwindale, CA 91706
www.hemosure.com

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