

# HEMOSURE® AccuREADER® A100

Fecal Immunochemical Test

## USING THE FIT

**Mailer Includes:** 1 Quick Guide, 1 Prepacked Return Mailer, 1 Sample Collection Tube, 2 Sample Collection Papers, 1 Specimen Pouch

**Collection Instructions:** No preparation is required and there are no dietary restrictions.



**1**

**! IMPORTANT!**  
Write your **name, date of birth, and date of collection** on the sample collection tube.

**2**

Lay sample collection paper across toilet bowl. Secure adhesive tabs and then lower toilet seat.

**3**

Deposit stool on collection paper.

**4**

Open the sample collection tube.

**5**

Poke grooved applicator into stool at **6 (SIX)** different locations.  
**Do NOT scoop stool. Only collect enough stool to cover the grooved applicator.**

**6**

Close the sample collection tube.  
**Do NOT stuff stool into the tube.**

**7**

Insert sample collection tube into specimen pouch. Then insert specimen pouch into return mailer and seal.

Return Address Required  
From: \_\_\_\_\_  
John Doe  
123 Main Street  
Anytown, USA 12345  
Contents: Diagnostic Specimens

Return To: \_\_\_\_\_  
Your Laboratory  
456 Main Blvd  
Anytown, USA 12345

**8**

Return immediately via mail or in person at your sample drop off location.

**Results may be invalid if:**

- Too much stool is collected
- Sample is frozen or overheated
- Feces touches water or urine in the toilet bowl
- The foil at the tube's base is damaged

**For assistance with these instructions, please contact:**

- Your physician's office  
OR
- Hemosure Technical Support:  
1-888-HEMOSURE (436-6787)  
E-mail: techsupport@hemosure.com